

Working with us Digitally

In addition to analog impressions, we accept digital impressions from all major intra-oral scanners. Below you will find the information and direction needed to add our lab to your scanner.

3Shape Trios

1. Go to us.3shapecommunicate.com in a web browser.
2. Connect with us by searching for Confident Dental Lab.
3. After connecting to us as a lab, select Confident Dental Lab when sending files.

Align iTero

1. Call 800-577-8767. Select Option 1.
2. Request that Confident Dental Lab is added to your scanner, and identify our lab using our phone number: 763-789-1121 or our lab ID # 24403
3. After Confident Dental Lab has been added, restart your scanner.
4. After connecting to us as a lab, select Confident Dental Lab on your scanner when sending files.

Dentsply Sirona CEREC Omnicam & Primescan

1. Have clinician log in to Connect Case Center on Omnicam or Primescan
2. Once logged in, navigate to the left hand side of the screen. Under "Partner" select "My Favorite Contractors"
3. In the middle of the screen titled "My Favorite Recipients" will appear. Scroll down to find a search bar for recipients you are connected to.
4. To find a new recipient, select "Search Direct" and type in Confident Dental Lab
5. Scroll down on results page, locate Confident Dental Lab and select the orange + sign on the right hand side of the page.
6. You are now connected to Confident Dental Lab and are able to send scans directly through the Connect Case Center

Medit i500 & i700

1. Sign up for your Medit Link Account by visiting MeditLink's registration page.
2. Request Confident Dental Lab as a partner and we will accept the request.
3. Send your cases directly through Medit Link.

Other Intra-Oral Scanners

1. Export STL file after completing your patient scan.
2. Email STL files and Digital Rx to tech@confidentlabmn.com